



### PRODUCT DESCRIPTION

The LTA Check ✓ Kit is a system of reagents used to verify the operation of Light Transmission Aggregometers and to provide instrument system quality control in the laboratory. FOR PROFESSIONAL USE ONLY.

### INTENDED USES

#### Performance Verification, Aggregation:

The LTA Check ✓ Kit is used to verify Light Transmission Aggregometer system performance.

#### Operator Training and Performance Verification:

The LTA Check ✓ Kit may be used to train operators to properly perform platelet aggregation and to document operator proficiency.

### PRINCIPLE

System functionality is determined by the ability of a plasma and ristocetin to induce agglutination of a standardized platelet suspension. The agglutination may be defined by two parameters: Aggregation and Slope (rate of reaction). These parameters when generated on a periodic basis may be used to evaluate the aggregation test system in the laboratory. Evaluation of the Aggregation and Slope of a test run using the materials provided on a periodic basis will generate information as to the functionality of the instrument test system. Ranges and limits for quality control and system functionality may be recorded by each laboratory and these results may be used to verify the functionality of the system on a "Channel to Channel" and a "Performance to Performance" basis. The laboratory is responsible for the establishment of the performance ranges of each lot and for each instrument in the laboratory.

### PRECAUTIONS

The LTA Check ✓ Kit is for RESEARCH USE ONLY (RUO) AND NOT FOR INJECTION OR INGESTION. The plasma and platelets have been tested at the source and found to be negative for HIV-1Ag, anti-HIV-1/2, Hepatitis B surface antigen, Hepatitis C antibody, Human T-Lymphotropic Type I and II (anti-HTLV I/II) and negative by a serological test for Syphilis. All plasma and platelets of human origin should be handled as being potentially hazardous.

### MATERIALS PROVIDED

1. Reagent 1, 1 X 0.5 mL
2. Reagent 2, 1 X 4.0 mL
3. Reagent 3, 1 X 1.0 mL
4. LTA Diluent, 1 X 10.0 mL, ready to use.
5. Ultrapure Water, 1 X 1.0 mL.

### REAGENT STORAGE

Reagent kits are stored at 2°C to 8°C prior to use. Stored reagent must be brought to room temperature prior to use.

**Upon completion of testing, all prepared, unused reagents must be discarded.**

### MATERIALS REQUIRED BUT NOT PROVIDED

1. Light Transmission Aggregometer (LTA)
2. Pipettors and tips
3. Disposable Stir Bars
4. Siliconized test tubes or cuvettes

### RECONSTITUTION

Allow vials to warm to room temperature before reconstitution. Reconstitute the vials as follows:

1. Add 0.5mL Ultrapure Water to Reagent 1.
2. Add 4.0mL LTA Diluent to Reagent 2.
3. Add 1.0mL LTA Diluent to Reagent 3.

Wait 5 minutes and then invert each vial to assure that all contents are rehydrated and allow the vial to stand an additional 25 minutes. Prior to use invert the vial an additional time to assure complete mixing and rehydration.

### PREPARATION OF THE AGGREGOMETER BLANK

The blank is prepared by mixing 1 part of Reagent 2 with 1 part of LTA Diluent. Volumes used are determined by the requirements of the instrument being used. **AFTER PREPARING BLANK LET STAND FOR 10 MINUTES BEFORE USE. ALWAYS REMIX BLANK IMMEDIATELY PRIOR TO USE.**

For full volume (500 µL)		For half volume (250 µL)	
<b>Reagent 2</b>	LTA Diluent	<b>Reagent 2</b>	LTA Diluent
250 µL	250 µL	125 µL	125 µL

### PERFORMING TESTS

The user must follow the test instructions provided by the instrument manufacturer.

### REAGENT USAGE IS IN THE FOLLOWING PROPORTION

Aggregation for full volume (500 µL)	Aggregation for half volume (250 µL)
50 µL Reagent 1	25 µL Reagent 1
400 µL Reagent 2	200 µL Reagent 2
50 µL Reagent 3	25 µL Reagent 3

**Testing must be completed within 45 minutes of reagent preparation.**

1. Place the appropriate number of test tubes required for testing into the incubation wells
2. Add a stir bar.
3. Incubate test tubes for one minute.
4. Add Reagent 1 into pre-warmed test tube for each test.
5. Add Reagent 2 into each tube, being careful to avoid splashing or the introduction of air bubbles.
6. Incubate the sample at 37°C for 2 minutes with stirring.
7. Set the 100% baseline (Blank) as required by the instrument .
8. Place the test tube mixture into the test well and add Reagent 3.
  - a. Do not allow reagent to run down the side of the tube.
  - b. Be careful not to spike the contents of the tube with pipetting technique.
9. Allow the test to run for 5 minutes and STOP when complete.

## EXPECTED RESULTS

Because the intended use is a verification of the LTA system performance, expected results should be evaluated on precision and reproducibility between test parameters (final aggregation (FA) and slope (PS)). The operator's proficiency may also be evaluated on the precision and reproducibility of the test results. Expected precision ranges are shown below.

		Expected CV (vs. average of established range)
* Expected Slope	25-50	± 10%
* Expected Agglutination	60-110%	± 10%

\* Performed on Bio/Data Corporation's Platelet Aggregation Profiler, Model PAP-8E. Other instruments or models may show different results.

NOTE: LTA Check✓ Performance Monitoring Software C/N 107149 is available for purchase. This software program accepts data generated from the aggregometer, calculates and prepares printable Levey-Jennings charts to evaluate and review. The Levey-Jennings chart is a graphical method used to display control or similar results for visual assessment of whether a process is in or out of specified limits as established by the laboratory. Utilizing Westgard analysis, a common set of statistical control rules will maximize error detection and minimize false rejections.

## Bibliography/Reference List

1. Westgard JO, Carey RN, Wold S. Criteria for judging precision and accuracy in method development and evaluation. Clin Chem 1974;20:825-833
2. Westgard JO, deVos DJ, Hunt MR, Quam EF, Carey RN, Garber CC. Method evaluation 1978 American Society for Medical Technology Bellaire, TX.
3. Westgard JO, Groth T, Aronsson T, Falk H, deVerdier C-H. Performance characteristics of rules for internal quality control: probabilities for false rejection and error detection. Clin Chem 1977;23:1857-1867.
4. Westgard JO, Hunt MR. Use and interpretation of common statistical tests in method comparison studies. Clin Chem 1973;19:49-57.

## PRODUCT AVAILABILITY

PRODUCT	NET CONTENTS	CATALOG NUMBER
Platelet Aggregation Profiler PAP-8E		106075
LTA Check✓ Performance Monitoring Software		107149
MagneTube <sub>™</sub> (Micro tube with stirbar) 50 sets/box		107146
Test Tubes, 7.25x55mm, Siliconized Micro 100/box		101521
Test Tubes, 8.75x50mm, Siliconized Macro 100/box		100336
Stir Bars, Plastic Coated Micro 2 tubes, 25pcs/tube		105990
Stir Bars, Plastic Coated Macro 2 tubes, 25pcs/tube		105977
Pipette, BioHit eLine Electronic 300µL		106232
Pipette Tips, BioHit 300µL, Tower Refill 10x96		106554
Pipette Tips, BioHit 300µL, Single Tray 10x96		106612
Lyophilized Platelets 3 x 4mL		101595
Lyophilized Platelets 1 x 10mL		101258
ADP 3 x 0.5mL		101312
Arachidonic Acid 3 x 0.5mL		101297
BETA/Pak® (ADP, Collagen, Ristocetin) 1 x 0.5mL each		101580
Collagen 3 x 0.5mL		101562
Epinephrine 3 x 0.5mL		101311
PAR/Pak®II (ADP, Collagen, Epinephrine) 2 x 0.5mL each		101310
Ristocetin		
AggRecetin® 1.5mg/mL 15mg		100968
AggRecetin 1.0-1.5mg/mL 15mg		100970
AggRecetin 100mg 100mg		101241
vW Select Kit 15 Determinations		107056
vW Select Kit 50 Determinations		106730
vW Factor Assay® 20 Determinations		103025
vW Abnormal Control Plasma 3 x 0.5mL		101270
vW Normal Reference Plasma 3 x 0.5mL		101269
vW Normal Control Plasma 3 x 0.5mL		106426

THIS PRODUCT IS WARRANTED TO PERFORM AS DESCRIBED IN THE LABELING AND IN THE LITERATURE OF BIO/DATA CORPORATION AND BIO/DATA CORPORATION DISCLAIMS ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR ANY OTHER PURPOSE AND IN NO EVENT SHALL BIO/DATA CORPORATION BE LIABLE FOR ANY CONSEQUENTIAL DAMAGES ARISING OUT OF AFORESAID EXPRESSED WARRANTY.



155 Gibraltar Road, Horsham, PA 19044 U.S.A.  
(800) 257-3282 U.S.A. (215) 441-4000 Worldwide  
(215) 443-8820 Fax Worldwide  
E-mail: [customer.service@biodatacorp.com](mailto:customer.service@biodatacorp.com)  
Internet: [www.biodatacorp.com](http://www.biodatacorp.com)



EMERGO EUROPE, Molenstraat 15, 2513 BH The Hague, The Netherlands

