**English****PRODUCT DESCRIPTION**

The MagneTube is a combination of 1 micro test tube and 1 micro stir bar conveniently packaged for immediate use in platelet function studies.

**Instructions for Use (for additional information refer to the PAP-4 or PAP-8E Operations Manual):**

Disposable Micro Test Tubes 7.25 x 55 mm containing Micro Stir Bars are for use with the PAP-8E aggregometer or PAP-4 series aggregometer using micro volume adapters.

Remove platform with tubes from box. Remove and discard shrink wrap. Unused MagneTubes should be kept in the platform and stored in the original box with the lid closed to prevent contamination.

**PRECAUTIONS**

MagneTubes are for **PROFESSIONAL LABORATORY USE AND IN-VITRO DIAGNOSTIC USE ONLY AND ARE NOT FOR INJECTION OR INGESTION.**

**NOTE TO USER:** *Any serious incident that occurs in relation to this device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.*

**MATERIALS PROVIDED**

Each box of MagneTubes contains 50 Silicized micro test tubes each preloaded with one plastic coated micro stir bar.

**Store at -20° to 40° C  
Single Use Only**

For a complete list of available products please go to our web site [www.biodatacorp.com](http://www.biodatacorp.com) or contact customer service below.

THE BIO/DATA CORPORATION PRODUCT LINE INCLUDES GENERAL PURPOSE, PROFESSIONAL LABORATORY USE REAGENTS INTENDED TO INDUCE AND REPORT PLATELET FUNCTION ACTIVITY AND RESPONSES. THIS PRODUCT IS WARRANTED TO PERFORM AS DESCRIBED IN ITS LABELING INCLUDING THE INSTRUCTIONS FOR USE. BIO/DATA CORPORATION MAKES NO CLAIM OR WARRANTY, EXPRESSED OR IMPLIED, OF THE CAPABILITY, FITNESS, OR MERCHANTABILITY FOR ANY OTHER PURPOSE. IN NO EVENT SHALL BIO/DATA CORPORATION BE LIABLE FOR ANY CONSEQUENTIAL DAMAGES ARISING OUT OF AFORESAID EXPRESSED WARRANTY.

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155 Gibraltar Road, Horsham, PA 19044 U.S.A.
+1 (800) 257-3282 U.S.A. +1 (215) 441-4000 Worldwide
Alpha Laboratories Ltd, 40 Parham Drive, Eastleigh, Hampshire, SO50 4NU United Kingdom
mdi Europa GmbH, Langenhagener Str. 71, 30855 Langenhagen, GERMANY